

Caster Wheel Replacement

Tools:

- 1-Wrench
- 2- Loctite- Thread locker (Optional)

Instruction


1- Place wrench as shown and turn the hex nut in counter clock wise direction to remove the wheel.

2- Clean threaded area free of dirt or residue.

3- If available, apply two drops of “loctite” thread locker on the thread (recommended).

4- Place new wheel and turn clockwise direction by hand.

5- Continue to tighten the nut with wrench provided.

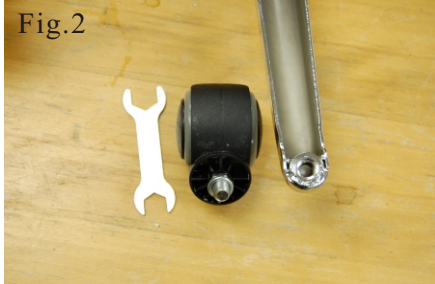

Gas Lift Cylinder Replacement

Required Tools:

- 1- Vise grip locking plier,
- 2- Rubber Mallet

Instruction

1- Using Vise grip plier to lock tightly onto top extended portion of the cylinder as shown Fig.1 and Fig.2

2- Turn plier handle as indicated to loosen the cylinder from the stool's bottom mounting plate.

3- Spray WD-40 lubricant between the cylinder body and the leg frame as indicated Fig.3

4- Turn leg frame up side down and tap the bottom of the cylinder with rubber mallet to loosen (Fig.4).

